# Suporte de Flash Duplo

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# Dual Flash Bracket

## USE INSTRUCTIONS

The **DUAL FLASH BRACKET** is light and resistant, it can be used with cameras that use either external or wireless flashes. The major advantage of the product is to offer the professional the possibility of better lighting angles to improve the quality of the photographs.

**INDICATIONS:** Used as a bracket for the camera and flashes during the taking of photographs.

**SF2-N:** Angulated model. Flash fitting recommended for the Nikon SB R200 model.

**SF2-C:** Flat model. Flash fitting recommended for any other application, except Nikon-manufactured devices. For Canon flash devices, it is recommended the model MT 24 Ex (standard).

**CLEANING:** Use clean cloth with neutral detergent. Dry thoroughly.

**STORAGE:** Store the products in their original and dated packaging, in a clean, dry place and protected from direct light.

**COMPOSITION:** Anodized aluminum, polyacetal, steel and rubber.

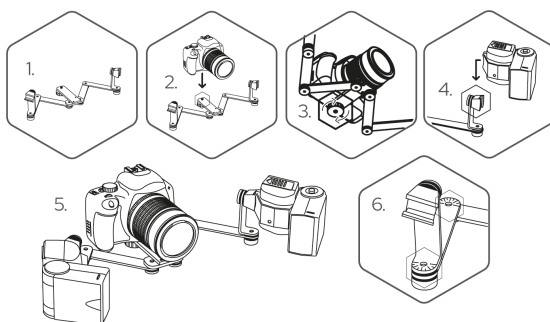
**PRECAUTION:** The useful life of the Flash Duplo Indusbello Support depends on two precautions taken during its use. Read the "how to use" of the product before using it.

- Avoid falls.
- Do not disassemble the product.

**DISPOSAL:** This product disposal should be done according to the current local environmental legislation.

## HOW TO USE:

- (1) Stretch the Dual Flash Bracket;
- (2) Attach;
- (3) Attach the camera with the central fixation screw;
- (4) Fit the flashes on the ends of the bracket;
- (5) Position the articulated support arms as needed;
- (6) Use the measures for your convenience.

Tip: Rotate the flash mount with the flash attached to make it easier to move.

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