Tamborel

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Endo Clean Stand

USE INSTRUCTIONS

Endo Clean Stand is a device for accommodating rotary and manual endodontic files. Its height allows it to accommodate files of up to 31mm with better stability and without damaging them. Available in multiple colors. Comes with two Refills for Endo Clean Stand Indusbello.

COMPOSITION: Polypropylene, Pigments and Polyurethane Foam.

INDICATIONS: Used as support and storage for manual and rotary endodontic files during all phases of root canal treatment.

INSTRUCTIONS FOR USE: Place the Indusbello Endo Clean Stand Refill on the bottom base of the Indusbello Endo Clean Stand and fit the upper part. Insert the endodontic files.

CLEANING: The removal of dirt (organic and inorganic matter) must be done using enzymatic detergent (minimum of three enzymes) with dilution according to the manufacturer's instructions. Cleaning can be done manually or mechanically in an ultrasonic tank. After washing the material, rinse with plenty of water to remove any dirt and/or detergent residue. Carry out the visual inspection and if any remaining dirt is observed, repeat the cleaning process.

DISINFECTION: Keep the product immersed in 0.2% peracetic acid for ten minutes.

STERILIZATION: Dry the product with a clean, dry cloth. Store the material in suitable packaging for sterilization. Sterilization must be carried out by autoclave at a temperature of 121°C to 135°C, in accordance with the autoclave manufacturer's recommendations. The cleaning, disinfection and sterilization process must be carried out after each use.

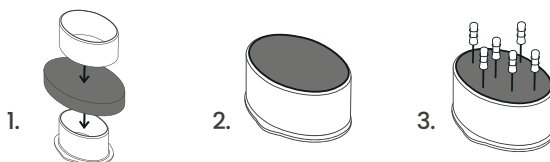
WARNING: The Refill and Stand must be sterilized separately before use. The manufacturer indicates that the Endo Clean Stand Refill must be discarded after use.

CAUTION: Do not sterilize the Endo Clean Stand with the refill attached. Do not touch the product to the inner walls of the autoclave.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

EXPIRY DATE: 05 years from the date of manufacture, in undamaged packaging. The shelf life of this product depends on the care taken during handling.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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