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5 MM

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USE INSTRUCTIONS

5 MM

DESCRIPTION: The steel matrix strip is flexible and adapts to the contour of the proximal faces of the teeth. This package contains a 50 cm roll of steel matrix strip, 5 mm high and 0.05 mm thick.

COMPOSITION: Stainless steel.

INDICATION: Indicated as a physical barrier for the restorative material and helps in the reconstruction of proximal contact.

INSTRUCTIONS FOR USE: Cut the Steel Matrix Strip in the size compatible with the tooth to be restored, place it in the matrix holder and take this set to the tooth you want to restore. Insert the wedge into the proximal embrasure for better matrix adaptation. After completing the procedure, remove the wedge and Steel Matrix Strip.

WARNING: Manufacturer recommends single use. Disposable product.

PRECAUTIONS: Product intended for use by dental professionals.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

VALIDITY: 5 years from the date of manufacture.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

7 MM

DESCRIPTION: The steel matrix strip is flexible and adapts to the contour of the proximal faces of the teeth. This package contains a 50 cm roll of steel matrix strip, 7 mm high and 0.05 mm thick.

COMPOSITION: Stainless steel.

INDICATION: Indicated as a physical barrier for the restorative material and helps in the reconstruction of proximal contact.

INSTRUCTIONS FOR USE: Cut the Steel Matrix Strip in the size compatible with the tooth to be restored, place it in the matrix holder and take this set to the tooth you want to restore. Insert the wedge into the proximal embrasure for better matrix adaptation. After completing the procedure, remove the wedge and Steel Matrix Strip.

WARNING: Manufacturer recommends single use. Disposable product.

PRECAUTIONS: Product intended for use by dental professionals.

PACKAGING: Pack the product in an exclusive place, closed cabinets and protected from any contaminants.

VALIDITY: 5 years from the date of manufacture.

DISPOSAL: The disposal of this product must be carried out in accordance with current local environmental legislation.

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